# Blade<sup>XT</sup>

## User Guide

BXTAG1SF–BXTAG9SF  
BXTAG1SM–BXTAG9SM

EN	User Guide	2
FR	Guide de l'Utilisateur	3
DE	Bedienungsanleitung für Anwender	4
IT	Guida per il Paziente	5
ES	Guia de Usuario	6
RU	Руководство пользователя	7

## Please keep these instructions.


The BladeXT is designed specifically for general sport and running.

### Steps and stairs

Because of the deep 'C' shape side profile of the Blade XT care should be taken when descending stairs and steps due to the potential hazard of 'over-stepping'.

Be aware of finger trap hazard at all times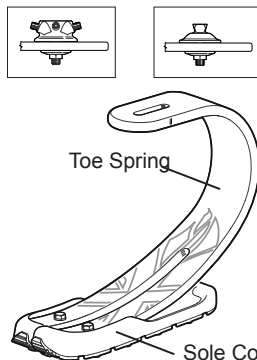

## Key points and precautions for use

Beware of slip hazard at all times especially on wet/smooth surfaces.

Excessive wear of the sole cover could lead to loss of grip.

Excessive wear or damage to the spring elements could lead to structural failure.

The limb should not be used if the sole cover is excessively worn, torn or loose.

Visually inspect the Blade XT especially the condition of the sole and toe spring before any activity.

Any changes in performance of this device must be reported to the practitioner.

Changes in performance may include:

- Reduced energy return
- Unusual noises
- Softening of the toe and/or heel

Any changes in body weight and/or activity level of the user must be reported to the Clinical Practitioner.

Assembly and repair of the BladeXT must only be carried out by suitably qualified Practitioners.

The product is water proof to a maximum depth of 1 metre.

Thoroughly rinse with fresh water after use in abrasive environments such as those that may contain sand or grit, for example, to prevent wear or damage to moving parts.

Thoroughly rinse with fresh water after use in salt or chlorinated water.

Exclusively for use between -15°C and 50°C.

### Cleaning:

Use a damp cloth and mild soap to clean outside surfaces, DO NOT use aggressive cleansers. Thoroughly dry before use.

### Manufacturer's Registered Address

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Blatchford SAS
Endolite India Ltd