# Multiflex SE

EN	Instructions for Use	2
FR	Instructions D'Utilisation	8
DE	Gebrauchsanweisung	14
IT	Istruzioni per L'Uso	20
ES	Instrucciones de Uso	26
NO	Bruksanvisning	32
RU	Инструкция протезиста	38
TR	Kullanım Talimatları	44

endolite

# 1 Description and Purpose

Multiflex SE combined with a Multiflex ankle provides the user with comfortable multi-axial motion, which enhances stability. The standard foot shell profile with sandal toe gives the foot a more natural appearance.

## Application

These instructions are for use by the practitioner.

The Multiflex SE foot is to be used exclusively as part of a lower limb prosthesis with a Multiflex ankle.

## Activity Level

This device is recommended for Activity Level 1–2 users.

There are exceptions to our recommendations and we want to allow for individual circumstances — there may be Activity Level

3–4\* users who may benefit from this product, but this decision should be made with sound and thorough justification.

\*Maximum user weight: 100 kg (220 lb).

## Contra-indications


This device may not be suitable for use in competitive sports events, as this type of user may be better served by a specially designed prosthesis optimized for their needs.

Intended for single user.

Ensure that the user has understood all instructions for use, drawing particular attention to [section 4](#) regarding maintenance.

 Appropriate footwear must be worn to avoid the risk of slipping in wet environments.

# 2 Construction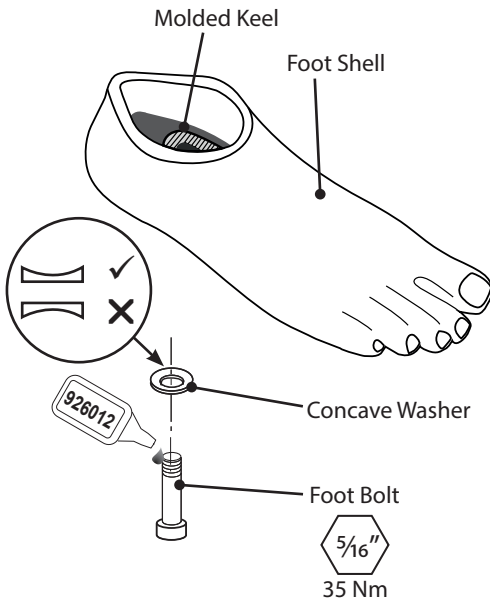

## Principal Parts:

- Foot Shell (PU)
- Molded Keel (fiber reinforced thermoplastic/aluminium alloy)
- Foot Bolt (stainless steel)
- Concave Washer (stainless steel)

**Note:** Use a new foot bolt when replacing the foot.

---

## 3 Function

Multiflex SE combines with a resilient multi-axial Multiflex ankle to give a smooth, stable walking action and allow some compliance to uneven surfaces. Multiflex SE comprises a lightweight, fiber reinforced thermoplastic keel which gives a gentle rolling action through stance. The keel lies within a standard foot shell profile and features a sandal toe for a more natural appearance. A range of heel heights can be accommodated via the serrated ankle interface.

---

## 4 Maintenance

Maintenance must be carried out by competent personnel.

It is recommended that the following maintenance is carried out annually:

- Check the foot bolt for tightness

Advise the user to report the following to the practitioner:

- Changes in body weight or Activity Level
- Changes in the performance of this device — changes may include:
  - Unusual noises or play

### Cleaning

Use a damp cloth and mild soap to clean outside surfaces. **Do not** use aggressive cleansers.

---

## 5 Limitations on Use

### Intended Life

A local risk assessment should be carried out based upon activity and usage.

### Lifting Loads

User weight and activity is governed by the stated limits. Load carrying by the user should be based on a local risk assessment.

### Environment

Avoid exposing Multiflex SE to corrosive elements such as water, acids and other liquids. Avoid abrasive environments such as those containing sand, as these may promote premature wear.


Suitable for outdoor use

Exclusively for use between -15 °C and 50 °C (5 °F and 122 °F).

**It is recommended that only Endolite products be used with this product.**

## 6 Bench Alignment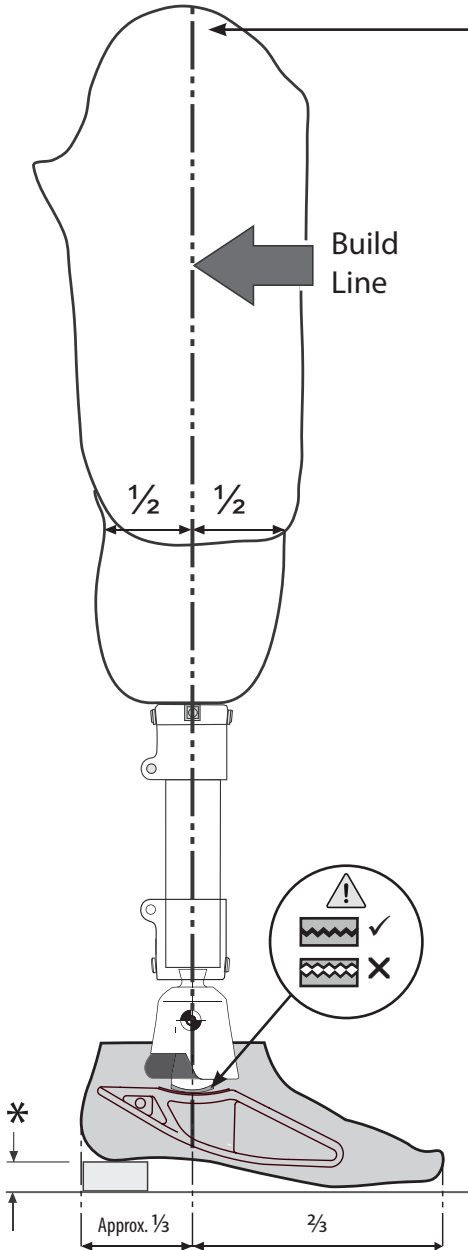

Align trans-femoral devices according to fitting instructions supplied with the knee, keeping the build line relative to the Multiflex ankle, as shown.

### Static Alignment

With flexion accommodated the build line should pass through the center of the socket in the sagittal (A-P) plane and through the Multiflex SE foot, as shown. Heel height of the shoe should be accommodated.

### Dynamic Alignment

#### Coronal Plane

Ensure that M-L thrust is minimal by adjusting the relative positions of the socket and foot.

#### Sagittal Plane

Check for smooth transition from heel strike to toe-off. Ensure that when standing the heel and toe are evenly loaded and that both are touching the floor.


⚠ Check that the serrations are engaged before tightening the foot bolt.

\* Allow for users own footwear.

# 7 Bonding a Cosmesis

## Before you begin


Lightly abrade and clean the mating surfaces with a suitable solvent cleaner.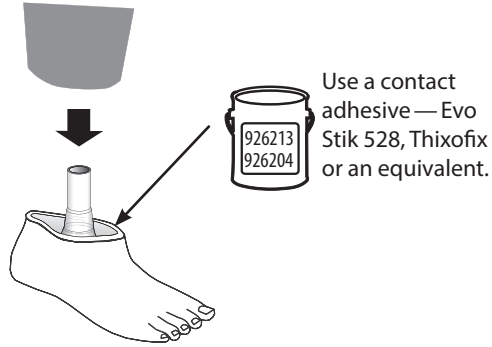

# 8 Technical Data

Operating and Storage Temperature Range:	-15 °C to 50 °C (5 °F to 122 °F)
Component Weight [size 26]:	425 g (15 oz)
Activity Level:	1–2
Maximum User Weight:	125 kg (275 lb)
Build Height:	40 mm
Heel Height:	10 mm
Range of Heel Height Adjustment:	± 15 mm (3x5 mm increments)

## Fitting Length

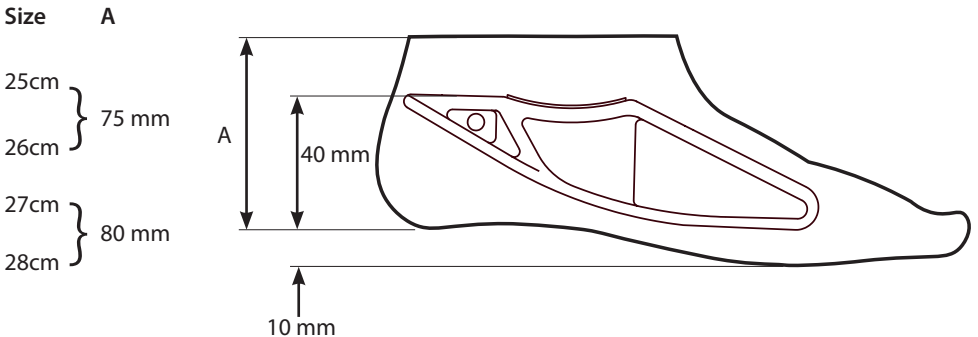---

## 9 Ordering Information

### Multiflex SE

Size (cm)	Part Number — Left	Part Number — Right
25	519525	519526
26	519527	519528
27	519529	519530
28	519531	519532

### Replacement parts

Part	Part Number
Foot bolt	400022
Concave washer	500021

**Liability**

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

**CE Conformity**

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class I Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

**Warranty**

Multiflex SE is warranted for 24 months. The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions. See Endolite website for the current full warranty statement.

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Endolite India Ltd
Endolite Deutschland GmbH,