Multiflex Standard

519119-519136

Multiflex Slim

519141-519148

Instructions for Use

EN	Instructions for Use	3
FR	Instructions D'Utilisation	9
DE	Gebrauchsanweisung	15
IT	Istruzioni per L'Uso	21
ES	Instrucciones de Uso	27
NO	Bruksanvisning	33
RU	Инструкция протезиста	39
TR	Kullanım Talimatları	45

Blatchford:

1 Description and Purpose

The Multiflex foot and ankle provide the user with comfortable multi-axial motion, which enhances stability. The low profile design with sandal toe gives the foot a more natural appearance.

Application

These instructions are for the practitioner.

The Multiflex foot is to be used exclusively as part of a lower limb prosthesis with a Multiflex ankle

Activity Level

This device is recommended for Activity Level 1–2 users.

There are exceptions to our recommendations and we want to allow for individual circumstances. There may be Activity Level 3–4* users who may benefit from this product,

but this decision should be made with sound and thorough justification.

*Maximum user weight: 100 kg (220 lb).

Contra-indications


This device may not be suitable for use in competitive sports events. This type of user may be better served by a specially designed prosthesis optimized for their needs.

Intended for single user.

Make sure that the user has understood all instructions for use, drawing particular attention to [section 4](#) regarding maintenance.

 Appropriate footwear must be worn to avoid slipping in wet environments.

2 Construction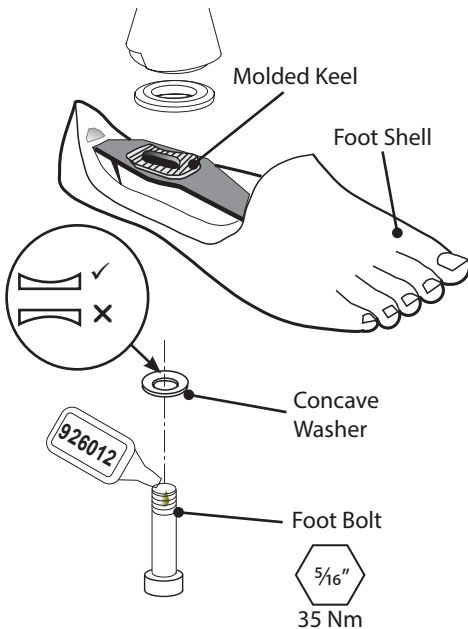

Principal parts:

- Foot Shell (PU)
- Molded Keel (fiber reinforced thermoplastic/aluminium alloy)
- Foot Bolt (stainless steel)
- Concave Washer (stainless steel)

Note: Use a new foot bolt when replacing the foot.

3 Function

The Multiflex foot combines with a resilient multi-axial Multiflex ankle to give a smooth, stable walking action and allow some compliance to uneven surfaces. The Multiflex foot comprises a lightweight, fiber reinforced thermoplastic keel which gives a gentle rolling action through stance and features a sandal toe for a more natural appearance — ideal for lower level activity users. A range of heel heights can be accommodated via the serrated ankle interface. Multiflex Slim features additional heel height capacity for a higher heel.

4 Maintenance

Maintenance must be carried out by competent personnel.

It is recommended that the following maintenance is carried out annually:

- Check the foot bolt for tightness

Advise users to report the following to their medical practitioner:

- Changes in body weight or activity level
- Changes in the performance of this device, for example:
 - Unusual noises or play

Cleaning

Use a damp cloth and mild soap to clean outside surfaces. **Do not** use aggressive cleansers.

5 Limitations on Use

Intended Life

A local risk assessment should be carried out based upon activity and usage.

Lifting Loads

User weight and activity is governed by the stated limits. Load carrying by the user should be based on a local risk assessment.

Environment

Avoid exposing the Multiflex foot to corrosive elements such as water, acids and other liquids. Avoid abrasive environments such as those containing sand, as these may promote premature wear.


Suitable for outdoor use

Exclusively for use between -15 °C and 50 °C (5 °F and 122 °F).

We recommend using Blatchford products with this device.

6 Bench Alignment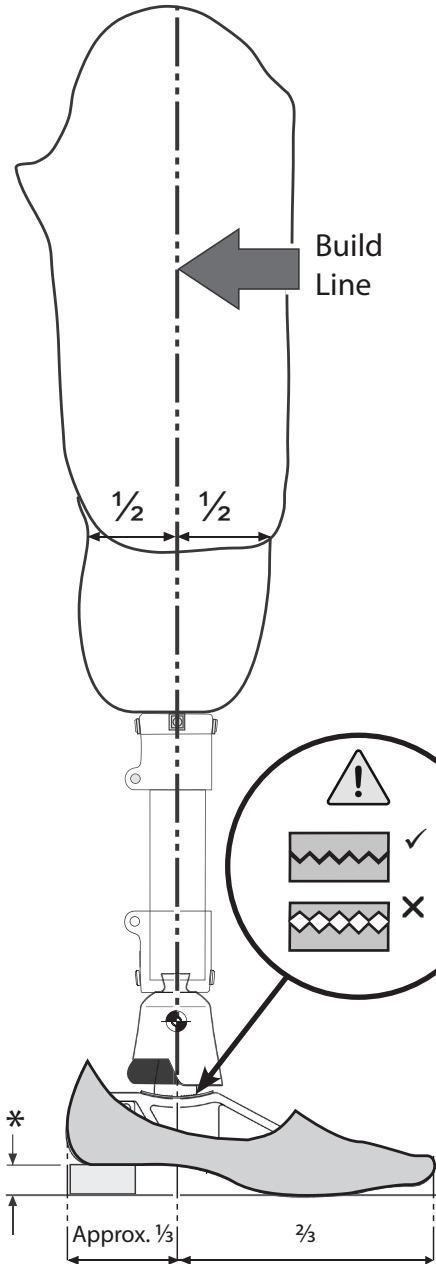

Align trans-femoral devices according to fitting instructions supplied with the knee, keeping the build line relative to the Multiflex ankle, as shown.

6.1 Static Alignment

With the flexion accommodated the build line should pass through the center of the socket in the sagittal (A-P) plane and through the Multiflex foot, as shown. Heel height of the shoe should be accommodated.

6.2 Dynamic Alignment

Coronal Plane

Make sure that M-L thrust is minimal by adjusting relative positions of socket and foot.

Sagittal Plane

Check for smooth transition from heel strike to toe-off. Also, make sure that when standing, the heel and toe are evenly loaded and that both are touching the floor.

Check that the serrations are engaged before tightening the foot bolt. Incorrectly engaged serrations can cause noise, looseness, and the eventual failure of the foot bolt.


* Allow for the user's own footwear

7 Technical Data


Operating and Storage Temperature Range:	-15 °C to 50 °C (5 °F to 122 °F)
Component Weight (size 26):	375 g (13.2 oz)
Activity Level:	1-2
Maximum User Weight:	125 kg (275 lb)
Build Height:	40 mm
Heel Height:	Multiflex Slim — 25 mm Multiflex Standard — 20 mm
Range of Heel Height Adjustment:	± 15 mm (3x5 mm increments)

Fitting Length

Multiflex Standard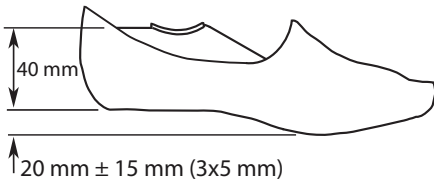

Multiflex Slim

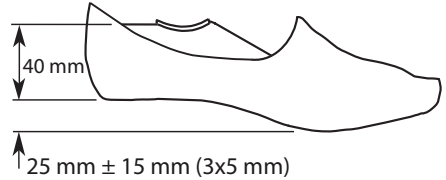8 Ordering Information

Multiflex Standard

Size (cm)	Part Number — Left	Part Number — Right
22	519119	519120
23	519121	519122
24	519123	519124
25	519125	519126
26	519127	519128
27	519129	519130
28	519131	519132
29	519133	519134
30	519135	519136

Multiflex Slim

Size (cm)	Part Number — Left	Part Number — Right
22	519141	519142
23	519143	519144
24	519145	519146
25	519147	519148

Part	Part Number
Foot Bolt	400022
Concave Washer	500021

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

CE Conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty

The Multiflex foot is warranted for 24 months.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licences and exemptions.

See the Blatchford website for the current full warranty statement.

Manufacturer's Registered Address

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Blatchford SAS
Endolite India Ltd