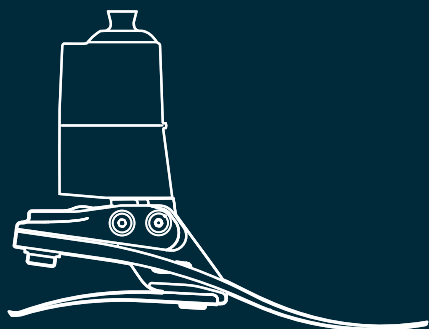
Elan

User Guide

ELAN22L1S—ELAN30R8S
ELAN22L1SD—ELAN30R8SD


EN	User Guide	1
FR	Guide de l'Utilisateur	7
DE	Bedienungsanleitung für Anwender	13
IT	Guida per il Paziente	19
ES	Guia de Usuario	25
NO	Brukerhåndbok	31
RU	Руководство пользователя	37
TR	Kullanım Kılavuzu	43

The Elan is a biomimetic foot and ankle system with microprocessor controlled dorsiflexion and plantar flexion. It is designed to adapt dynamically (via motorized valves) to provide assistance when walking on a variety of surfaces, when changing gait speeds and increased levels of safety and security when walking and standing.

Features

- Dynamic damping resistance aims to improve gait characteristics enabling safer walking
- Microprocessor control of resistance in real time
- The Elan Software adapts to individual user gait characteristics
- Standing Support mode
- Aims to reduce stress at the socket interface
- PC interface setup with Bluetooth® link and direct communications
- Power is supplied via an integral rechargeable battery pack

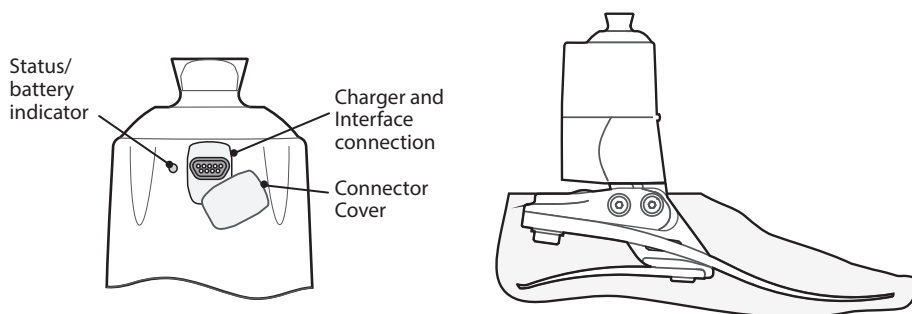This device provides limited self-alignment of the prosthesis on varied terrain and following footwear changes. It is intended to improve postural sway, balance stability and inter-limb loading symmetry while easing high pressures at the socket interface.

Safety Information

-  **The caution symbol highlights safety information which must be followed carefully.**
-  1. Any changes in performance of the ankle motion e.g. restricted movement, non-smooth motion should be immediately reported to your service provider.
 -  2. Always use a hand rail when descending stairs and at any other time if available.
 -  3. Any excessive changes in heel height after programming will adversely affect ankle function and should be immediately reported to your service provider for reprogramming and calibration.
 -  4. After continuous use the ankle casing may become hot to the touch.
 -  5. Avoid strong magnetic fields, sources of electrical interference, atmospheres containing liquids and/or powders.
 -  6. Avoid exposure to extreme heat and/or cold.
 -  7. The electronic components are sealed for water resistance. However the Elan is not intended for use when immersed in water or as a shower prosthesis. If the ankle comes into contact with water wipe dry immediately.
 -  8. Ensure no water enters the interface/charging cable connector.
 -  9. The Elan is not suitable for extreme sports, running or cycle racing, ice and snow sports, extreme slopes and steps. Any such activities undertaken are done so completely at the users' own risk. Recreational cycling is acceptable.
 -  10. Contact your service provider for advice if the Elan is to be used with a pacemaker or any other electronic medical device.
 -  11. Assembly, programming and repair of the Elan must only be carried out by suitably qualified practitioners, certified by Blatchford.
 -  12. The user must not adjust or tamper with the setup of the Elan.
 -  13. The user should be advised to contact their clinician if their condition changes.
 -  14. The Elan detects standing and walking on inclines, declines and walking at various speeds on normal ground. Non-identified movements such as treadmills and travalators may cause unexpected ankle behaviour.
 -  15. Never plug the charger into the Elan while still wearing the limb.
 -  16. Only charge the batteries in the Elan using the charger supplied with the product. Do not use this charger for charging any other device. Ensure the charging port connector cover is correctly refitted after charging, replace if damaged.
 -  17. Ensure only suitably retrofitted vehicles are used when driving. All persons are required to observe their respective driving laws when operating motor vehicles.
 -  18. Ensure charging cable and Bluetooth dongle are correctly aligned prior to connection and that adequate access to the charging port is provided (See Section 3).
 -  19. To minimise the risk of slipping and tripping, appropriate footwear that fits securely onto the footshell must be used at all times.
 -  20. Be aware of finger trap hazard at all times

2 Key points and precautions for use

Maintenance must be carried out by competent personnel.

It is recommended that maintenance is carried out annually and a service carried out as a condition of warranty every 20 months.

Key points:

Any changes in performance of this device must be reported to the practitioner
Changes in performance may include:

- Increase in ankle stiffness
- Reduced ankle support (freer movement)
- Any unusual noise

At low battery power levels the Elan will go into sleep mode, switch off dynamic control and use only the basic resistance settings. Charge the Elan batteries as soon as possible to restore Dynamic mode.

The practitioner must also be informed of the following:

- Any changes in body weight and/or activity level.
- Significant wear or excessive discoloration from long term exposure to UV.

Cleaning:

Use a damp cloth and mild soap to clean outside surfaces, DO NOT use aggressive cleansers. Ensure water/liquids do not enter the interface connector (Micro D).

Transport and Storage:

Storage temperature range between -15°C and 50°C (5°F and 122°F).

If the Elan is shipped, the packaging supplied with the unit should be used. When built into a limb it should be stored securely so as not to damage the unit.

Lifting loads:

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

Environment:

Avoid exposing the Elan to corrosive elements such as water, acids and other liquids. Also avoid abrasive environments such as those containing sand for example as these may promote premature wear.

Exclusively for use between -15°C and 50°C (5°F and 122°F).

Suitable for outdoor use

It is recommended that only Blatchford products should be used in conjunction with the Elan.

3 Battery Charging

i Please advise user of the charging information in this section.
The batteries are not field replaceable and will be replaced as necessary when serviced.

Read the user instructions supplied with the charger before using for the first time.

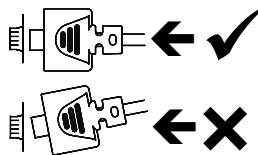
We recommend the Elan is charged daily; it should take 2 hours to charge fully, depending on residual charge. An exhausted or flat battery may take longer.

Lay the limb on a flat surface and use the Charger Cable Adapter to connect the charger into the socket in the top of the Elan body.

Use the indicator light on the charger to confirm that charging has initiated correctly:

Ensure correct alignment of the connector prior to connection to the charging port

LED	Charging state
Yellow	Initialisation state/no battery
Orange	Fast charge
Green/Yellow	Maintain/ Trickle charge
Green	Fully charged
Orange/Green	Error

The Elan will take several seconds to reset when the charger is disconnected and a single blue flash from the LED will confirm a correct startup.

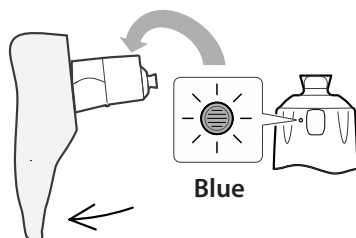
- !** 1. Only charge the batteries in the Elan using the charger supplied with the product. Do not use this charger for charging any other device.
- !** 2. The battery charger can be used with an electrical outlet that is rated between 100V and 240V AC, 50/60Hz.
- !** 3. Never plug the charger into the Elan while still wearing the limb.
- !** 4. During charging no power is delivered to the ankle control system.
- !** 5. The front of the ankle may become warm during charging but should never be too hot to touch. There should be no abnormal smell. If this is the case immediately unplug from electrical supply and contact your service provider.
- !** 6. Do not refit the limb until the charger has been disconnected.
- !** 7. Always recharge the Elan before use if it has not been used for an extended period of time.

- ⚠ 8. Ensure the connector cover is refitted after charging
- ⚠ 9. When charging an empty battery the charger may revert to trickle charge as a precaution to prevent damaging the battery.

Battery Charge/Status Indication

To check the battery charge status, tilt the foot down vertically and hold stationary for 4 seconds. The LED at the rear of the Elan will flash blue, 1-3 times (see table) to indicate the charge status and then repeat. Tilt the foot again to check the battery status again.

Note: this feature can be switched off, see Section 14 > Options.

LED	Battery Charge
3 flashes	Maximum
2 flashes	High
1 flash	Low

After charging the Elan the LED may indicate 2 or 3 flashes this will depend on the charging time, the age and condition of the battery.

Elan Status Indication

1 Single Flash

When the charger is disconnected the LED will flash once to indicate correct start up.

2 Continuous Flash

If the LED flashes continuously after disconnection this may indicate an error has been detected. To reset the Elan, reconnect and then disconnect the charger the Elan should flash once to indicate correct start up. If the Elan will not reset contact your service provider.

3 Alternate Long-Short Flash

Indicates Battery shutdown; If this is seen shortly after an adequate charging period, contact your service provider a battery service may be required.

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

CE Conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty

The Elan is warranted for – 36 months – foot shell 12 months - glide sock 3 months.

See Blatchford website for the current full warranty statement.

Scheduled service every 20 months is required, for maintenance of the warranty.

For extended warranty contact service provider.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licences and exemptions.

This device complies with part 15 of the FCC Rules.

Operation is subject to the following three conditions:-

- 1 This device may not cause harmful interference.
- 2 This device must accept any interference received, including interference that may cause undesired operation.
- 3 Contains pre-approved Bluetooth® module FCC ID: T9J-RN41-1

Environmental Aspects

This symbol indicates that the product contains electrical/electronic components and/or batteries that should not be disposed of in general waste or be incinerated at the end of the product's life.

At the end of the product's life, all electrical/electronic components and/or batteries should be recycled or disposed of in accordance with the current regulations for the handling of WEEE (Waste Electrical and Electronic Equipment), or equivalent local regulations. The remainder of the product should also be recycled where possible in accordance with local waste recycling regulations.

Manufacturer's Registered Address

Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH, UK.

Bluetooth® is the registered trademark of Bluetooth® SIG

Elan and Blatchford are registered trademarks of Blatchford Products Limited.

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© Blatchford Products Limited 2019. All rights reserved.
Endolite India Ltd.