# Senior

## Instructions for Use

SNR22L-SNR28R  
SNR22LD-SNR28RD

EN	Instructions for Use	3
FR	Instructions D'Utilisation	9
DE	Gebrauchsanweisung	15
IT	Istruzioni per L'Uso	21
ES	Instrucciones de Uso	27
NO	Bruksanvisning	33
RU	Инструкция протезиста	39
TR	Kullanım Talimatları	45

# 1 Description and Purpose

EN

## Application

These instructions are for the practitioner.

The term *device* is used throughout these instructions for use to refer to Senior.

Make sure that the user understands all the information in these instructions for use, particularly the maintenance and safety information.

This device is a foot unit that is for use as a component of a lower limb prosthesis.

This device is a lightweight foot with a cushioned heel and a sandal toe.


## Activity Level

This device is for Activity Level 1 and 2 users.

## Contra-indications

This device is for a single user.

# 2 Construction

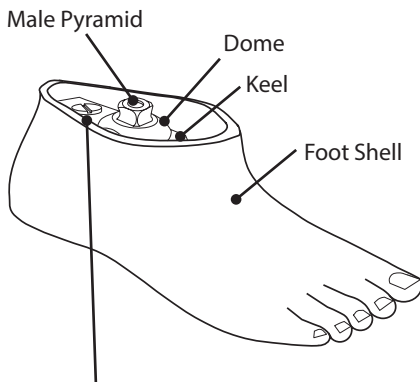M10x40 Attachment Screw

Principal parts:

- Foot Shell (PU)
- M10x40 Attachment Screw (nylon)
- Keel (fiber reinforced thermoplastic)
- Dome (aluminum alloy)
- Male Pyramid (stainless steel)

**Warning:** The attachment screw is **not** for adjustments.

# 3 Function

This device comprises an open foot shell, a heel cushion and a keel. The heel cushion dampens heel strike for comfortable walking. The resilient keel allows the foot to roll over smoothly into late stance.

---

## 4 Maintenance

Only competent personnel should carry out maintenance.

Carry out the following maintenance annually:

- Make sure that the attachment screw is tight and that the gap is closed between the foot shell and the keel. See [section 8 step 3](#) for instructions.
- Make sure that the attachment screw is covered with sealant to prevent tampering.

Advise the user to report the following to the practitioner:

- Changes in body weight or activity level
- Changes in the performance of this device, for example, unusual noises or play.

### Cleaning

Use a damp cloth and mild soap to clean outside surfaces. **Do not** use aggressive cleansers.

---

## 5 Limitations on Use

### Intended life

A local risk assessment should be carried out based upon activity and usage.

### Lifting loads

User weight and activity is governed by the stated limits. Load carrying by the user should be based on a local risk assessment.

### Environment

This device is waterproof up to a maximum depth of 1 meter.

To prevent wear or damage to moving parts, thoroughly rinse the device with fresh water after use in:

- Abrasive environments such as those that contain sand or grit
- Salt or chlorinated water

Finish the device to prevent water ingress (see [section 7](#)). If water enters the device, invert it to allow the water to drain, then leave it to dry.


Suitable for submersion

 Take care on wet and slippery surfaces.

Exclusively for use between -15 °C and 50 °C (5 °F and 122 °F).

We recommend using Blatchford products with this device.

## 6 Alignment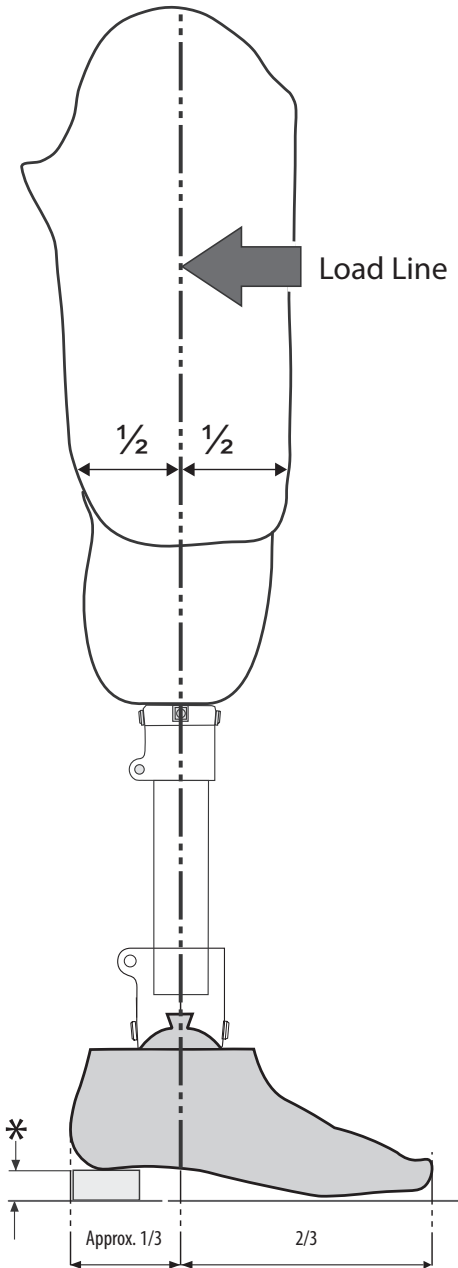

### Additional components

Follow the alignment instructions that are supplied with any additional components.

### Static alignment

#### Sagittal plane

With the flexion accommodated, the load line should pass through the center of the socket and through the center of the pyramid.

Make sure that, when the user is standing, the heel and the toe are evenly loaded and the foot is in full contact with the floor.

### Dynamic alignment

#### Coronal plane

Minimise M-L thrust by adjusting the relative positions of the socket and the foot.

#### Sagittal plane

Make sure that the user transitions smoothly from heel strike to toe-off.

\* Accommodate the heel height of the user's footwear


## 7 Bonding a Cosmesis

### Before you begin

Lightly abrade and clean the mating surfaces with a suitable solvent cleaner.


Use a contact adhesive—Evo Stik 528, Thixofix or an equivalent.

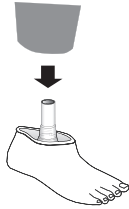## 8 Changing the Foot Shell

### Before you begin



Remove the anti-tamper sticker and the sealant from the attachment screw.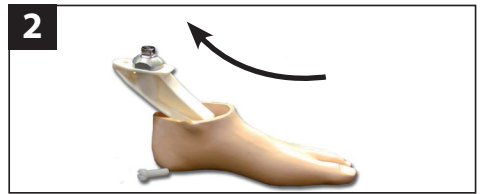

**3**  Wear protective gloves.

### Before refitting the keel

Apply grease (928016) to all the surfaces of the keel that come into contact with the inside of the foot shell.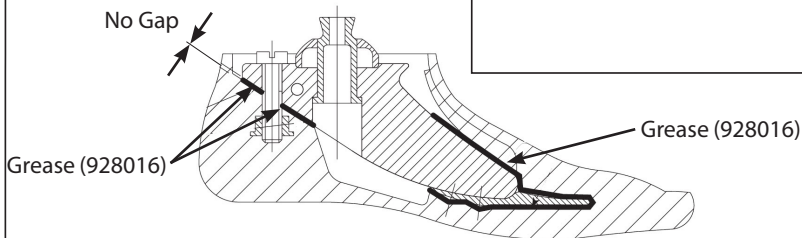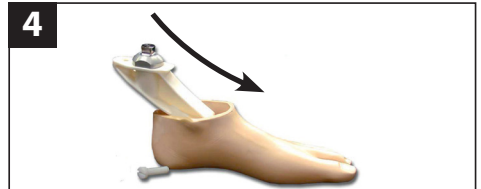

1. Press the keel firmly against the foot shell to close the gap between them.

 A gap between the keel and the foot shell can cause the keel to lift out of the foot shell.

2. Tighten the attachment screw until the screw head contacts the keel, then tighten a further  $\frac{1}{4}$  turn.


3. Cover the attachment screw with sealant to prevent tampering.

# 9 Technical Data

Operating and Storage Temperature Range:	-15 °C to 50 °C (5 °F to 122 °F)
Component Weight (size 26):	395 g (13.9 oz)
Activity Level:	1–2
Maximum User Weight:	125 kg (275 lb)
Proximal Alignment Attachment:	Male pyramid (Blatchford)
Range of Adjustment	±7 ° angular
Build Height:	70–80 mm
Heel Height:	19 mm

## Fitting length

Size (cm)	A
22	70 mm
23	
24	
25	75 mm
26	
27	80 mm
28	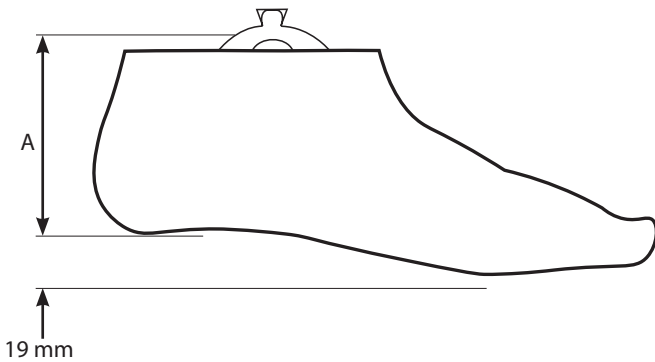


# 10 Ordering Information

## Order example

This device is available in sizes 22–28.

**Note:** For a dark-tone foot shell add a D to the end of the order number.

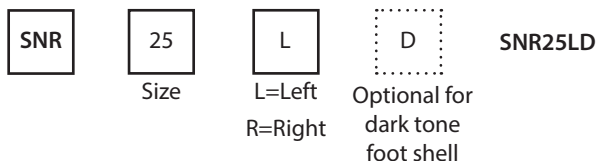## Foot Shell (add a D for dark tone)

Size (cm)	Left Part Number	Right Part Number
22	529701	529702
23	529703	529704
24	529705	529706
25	529707	529708
26	529709	529710
27	529711	529712
28	529713	529714

## Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

## CE conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class I Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

## Warranty

This device is warranted for 24 months. The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions. See Blatchford website for the current full warranty statement.

## Manufacturer's Registered Address

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Blatchford SAS
Endolite India Ltd