Linx

User Guide

LINX22L1S – LINX30R8S
LINX22L1SD – LINX30R8SD

EN	User Guide	1
FR	Guide de l'Utilisateur	12
DE	Bedienungsanleitung für Anwender	23
IT	Guida per il Paziente	34
ES	Guia de Usuario	45
NO	Brukerhåndbok	56
RU	Руководство пользователя	67
TR	Kullanım Kilavuzu	78
AR	دليل المستخدم	89

Contents	1
1 Description and Intended Purpose	2
2 Safety Information	3
3 Construction	4
4 Key Points	5
5 Transport and Storage	5
6 Battery Charging	6
6.1 Magnetic Charger Connection	8
6.2 Battery Status	8
7 User Modes	9

The term *device* is used throughout this manual to refer to Linx.

Warnings


Battery Safety

Linx contains Li-Ion type rechargeable batteries. Please read and comply with the safety information, see Section 2 and Section 6. Failure to do so may cause the battery to become hot, explode, or ignite and cause serious injury.

IMPORTANT: Battery Charging

To charge the device ensure that the On/Off Switch is in the **ON** position. When the device is charging the Battery Charger LED will be flashing green.

The battery should only be charged within the temperature range 10 °C to 40 °C (50 °F to 104 °F).

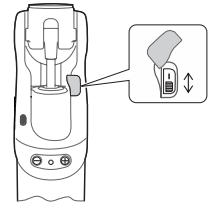Battery On/Off Switch

Low Temperature Operation

IMPORTANT: If the device is subjected to temperatures lower than -10°C (14°F) it must be returned to be inspected for possible damage to the battery pack.

Following extended periods of inactivity at temperatures below 0°C (32°F) the limb may swing less freely. In such an eventuality, it is recommended to exercise caution while descending stairs and avoid descending stairs leg-over-leg using the knee yield until you feel the swing returns to normal.

Magnetic Connector Cleaning

Regular cleaning of the magnetic charging surfaces is required to ensure no liquids, contamination or debris prevents correct connection. (See Section 4 for cleaning advice).

Notes:

1. The Linx firmware has been updated. To check which version of the Linx firmware is installed on your device, reset the device. If your device emits 2 short beeps 30 seconds after turning on, it is installed with version 1.0N or higher of the Linx firmware.
2. This symbol represents information that only applies to devices with version **1.0N or higher** of the Linx firmware: **1.0N+**
3. This symbol represents information that only applies to devices with version **1.0B or lower** of the Linx firmware: **1.0B**

Package Contents:

- 1 Linx limb
- 2 User guide
- 3 Cable bag and accessories
 - 1 Battery charger
 - 2 Battery charger AC power socket outlet adapter(s)

1 Description and Intended Purpose

The device is a fully integrated, microprocessor-controlled knee and foot system. The four microprocessors provide simultaneous control over both foot and knee co-ordinating their responses for a variety of terrains and situations in one complete biomimetic limb. Individual adjustment can be made for slopes and stairs and a stop and lock mode for standing.

Application

The device is to be used exclusively as part of a lower limb prosthesis.

Intended for single user.

Make sure that you understand all instructions for use, drawing particular attention to all maintenance and safety information sections.

Features

- All the functions of a microprocessor control in both knee and hydraulic ankle
- Centralised control of knee and foot to give the user a better response
- Proven to reduce stress at the socket interface
- An easy access single charging and point
- Longer battery life, 3 days average use from the internal rechargeable Li-ion battery pack

The enhanced stability system offers:

- *Stop and Lock* mode, control of knee flexion and ankle for standing
- Controlled ramp descent with *Braking* effect of knee and ankle
- Provides assistance during ramp ascent
- Dynamic stair descent with an easy start of knee flexion coupled with increasing support to gently lower the user during flexion
- Supportive resistance to flexion as soon as the knee stops flexing
- Increased supportive resistance to flexion to prevent 'stumbles' associated with instability at heel strike


2 Safety Information

-  The caution symbol highlights safety information which must be followed carefully.
-  Any changes in the performance or function of the limb e.g. restricted movement, non-smooth motion or unusual noises should be immediately reported to your service provider.
 -  Always use a hand rail when descending stairs and at any other time if available.
 -  Any excessive changes in heel height after programming will adversely affect limb function and should be immediately reported to your service provider for reprogramming and calibration.
 -  After continuous use the ankle and knee casing may become hot to the touch.
 -  Avoid strong magnetic fields, sources of electrical interference, atmospheres containing liquids and/or powders.
 -  Do not place near any heat source. Do not leave in direct sunshine or inside a car in hot weather.
 -  The device is not intended for use when immersed in water or as a shower prosthesis. If the limb comes into contact with water wipe dry immediately.
 -  Ensure no water enters the charger plug socket and the cover is on at all times.
 -  Before connecting the battery charger to the device, ensure that the magnetic mating surfaces of the charging connector and charging terminal are clean, dry and free of any debris or conductive materials that could prevent correct charging operation. (See further warnings in Section 6.1 and cleaning advice in Section 4.)
 -  Never connect the charger to the device while still wearing the device.
 -  The device is not suitable for extreme sports, running or cycle racing, ice and snow sports, extreme slopes and steps. Any such activities undertaken are done so completely at the users' own risk. Recreational cycling is acceptable.
 -  Be aware of finger trap hazard at all times
 -  Contact your service provider for advice if the device is to be used with a pacemaker or any other electronic medical device.
 -  Assembly, programming, maintenance and repair of the device must only be carried out by a suitably qualified clinician that has attended an approved training course.
 -  The user must not adjust or tamper with the setup of the device.
 -  The user should be advised to contact their clinician if their condition changes.
 -  The device detects standing and walking on inclines, declines and walking at various speeds. Non-identified movements may cause unexpected behaviour.
 -  Only charge the batteries in the device using the charger supplied with the product. Do not use this charger for charging anything else.
 -  Ensure only suitably retrofitted vehicles are used when driving. All persons are required to observe their respective driving laws when operating motor vehicles.
 -  Make sure that no liquids enter the device during donning or normal use.
 -  The charging connector and terminal are sources of strong magnetic fields. Always keep a safe distance (at least 10 cm) between these magnets and all objects that can be damaged or affected by magnetism (such as watches, pacemakers, credit cards, or any magnetically stored media).
 -  To minimise the risk of slipping and tripping, appropriate footwear that fits securely onto the footshell must be used at all times.

3 Construction

Principal Parts

- Knee Module including pyramid (aluminium/St. Stl./titanium, carbon)
- Ankle Module (aluminium/St. Stl.)
- Pylon (aluminium)
- Heel & Toe Springs (e-Carbon)
- Spring Attachment Screws (titanium/St.Stl.)
- Glide Sock (UHM PE)
- Foot Shell, cables and flexi-boards (polymers and PU)
- Batteries (lithium ion)

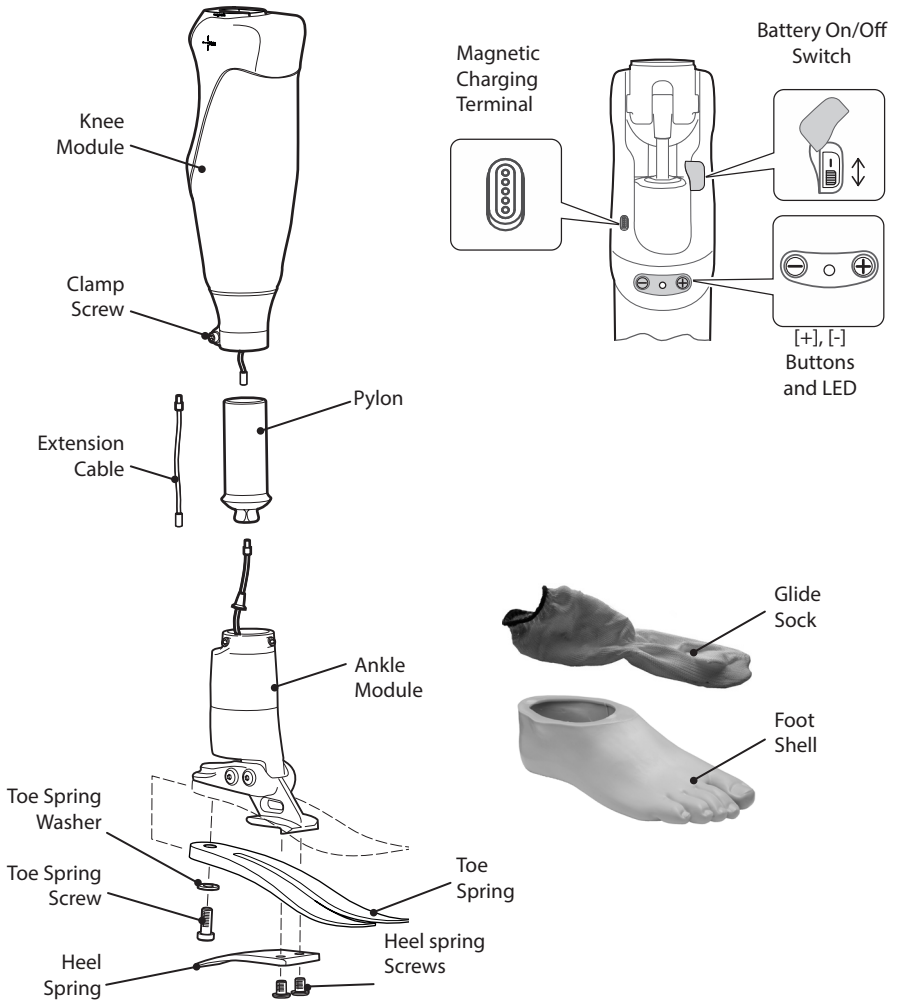4 Key Points

The device must not be used with large variations in heel height.

Do not attempt stairs descent leg-over-leg on this device unless you have been trained for stairs descent.

High flexion resistance settings may induce some noise during stairs descent, this will not affect function.

Any changes in performance of this device must be reported to the clinician.

Changes in performance may include:

- Increase or decrease in knee and/or ankle stiffness
- Instability
- Knee starting to extend too aggressively
- Reduced ankle/knee support (free movement)
- Any unusual noise
- Any unrecognized beeps or warning lights

The practitioner must also be informed of the following:

- Any changes in body weight and/or activity level
- Significant wear or excessive discoloration from long term exposure to UV

Cleaning:

Use a damp cloth and mild soap to clean outside surfaces, **DO NOT** use aggressive cleansers. Ensure water/liquids do not enter the charging socket. Thoroughly dry before use.

To clean the magnetic mating surfaces of the charging connector and charging terminal use a clean and dry cotton swab or a soft bristle brush. Make sure that no cotton fibres or bristles are left on the surfaces.

Never apply water or any other cleaning liquid to the magnetic mating surfaces.

Environment:

Avoid exposing the device to corrosive elements such as water, acids and other liquids. Also avoid abrasive environments such as those containing sand for example as these may promote premature wear.

The charging surfaces of the connectors should be kept away from other magnetic or conductive materials.

Suitable for outdoor use

Exclusively for use between -10 °C and 50 °C (14 °F to 122 °F).

5 Transport and Storage

1. Ensure that the device is in a straight (fully extended) position when not in use.
2. If the limb is to be stored upright then it must be secured to prevent it falling over or flexing.
3. If the device is shipped either on its own or built into a limb it should be maintained in an extended position and packaged in a suitable box to allow this.
4. If a unit is to be returned, contact Blatchford prior to sending if any obvious physical damage has occurred to the battery pack.
5. For transport purposes the unit must be switched off, and packed securely so that
 - a) it cannot be inadvertently switched on and become operative while in transit;
 - b) the unit is thoroughly protected from damage in transit.

6 Battery Charging


The device contains Li-ion type rechargeable batteries. Please read and comply with the safety information. Failure to do so may cause the battery to become hot, explode, or ignite and cause serious injury.

The batteries are not field replaceable and will be replaced as necessary when serviced.

If the limb is subjected to temperatures lower than -10°C (14°F) it must be returned to be inspected for possible damage to the battery pack.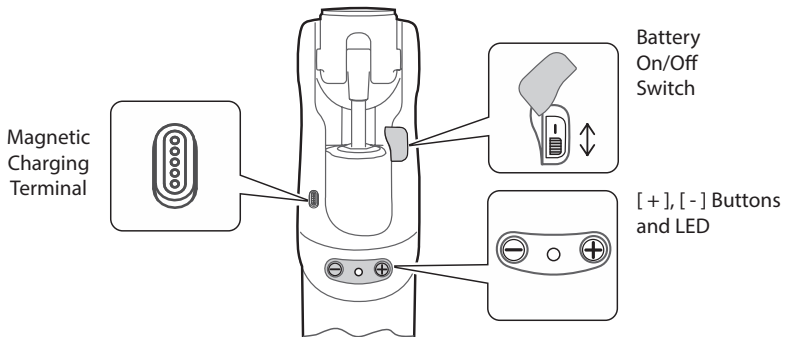

1. The battery should only be charged in temperatures between 10°C to 40°C (50°F to 104°F).

2. Only charge the batteries in the device using the charger supplied with the product. Do not use this charger for charging any other device.

3. The battery charger can be used with an electrical outlet that is rated between 100V and 240V AC, 50/60Hz.

4. Never plug the charger into the device whilst still wearing the device.

5. During charging no power is delivered to the device control system.

6. The knee module may become warm during charging but should never be too hot to touch. There should be no unusual smell, if this is the case immediately switch OFF[O] the battery, unplug the charger from the electrical supply and contact your service provider.

7. Do not refit the device until the charger has been disconnected.

8. Always recharge the device before use if it has not been used for an extended period of time.


6 Battery Charging (continued)

Fully charge the battery before first use.

We recommend that the device limb should be fully charged every night. It should take 8 hours to charge fully (from fully discharged). We also recommend battery charging whenever convenient.

Step-by-Step Guide:


1. Lay the limb on a flat surface.
2. Ensure the mating surfaces of the magnetic charging connector and terminal are clean (see Section 4).
3. Magnetically join the charging connector to the charging terminal at the posterior of the knee and check alignment is correct (see Section 6.1).
4. Switch the Battery On/Off switch to **ON [I]**.
5. Check the Battery Charger LED is flashing green to confirm the battery is charging.
6. Refer to table below to ensure Limb is charging.
7. Disconnect from the charger when charged sufficiently and refit terminal cover.
8. Wait for 30 seconds while observing the device LED changing color as indicated: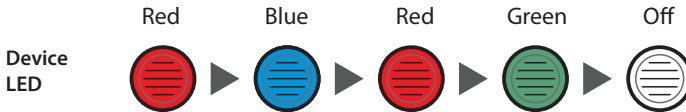


9. After a successful reset sequence, 2 short beeps will be heard (if this alert is not disabled; see* below).
10. If reset sequence is unsuccessful or the LED remains Red, switch the battery **OFF [O]** then **ON [I]** and repeat from step 8.

Battery Charger LED	Charging state
Yellow	Standby
Green > fast flashing	Fast charging
Green > slow flashing	Close to full charge
Green	Maintain (trickle charge)
Yellow > fast flashing	Error (Ensure Battery Switch is ON [I])

Note... If the charger LED is yellow and the Limb is making a clicking noise check that the Battery switch is in the **ON [I]** position.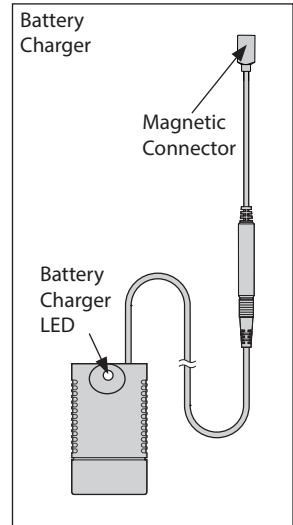

Note... Do not move or don the limb while reset is processing otherwise this will prolong the reset time.

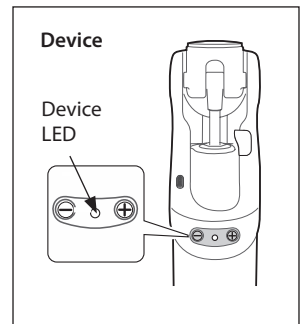1.0N+ You can enable or disable the audible reset confirmation by pressing and holding the [+] and [-] buttons simultaneously for 2 seconds. The change will be confirmed by 2 short beeps.


6.1 Magnetic Charger Connection

The charging connector can be connected to the device through the charging terminal located at the posterior of the knee.

The connector and the terminal magnetically lock together and correctly self-align on connection (Figure 1).


Note... The connector and the terminal are not polarized so orientation is not critical (Figure 2).

-  **Please check for any misalignment between mating surfaces as this may result in no charging. This can happen especially when the charging setup puts strain on the charging cable (Figure 3).**
-  **Before charging ensure charging surfaces are clean, dry and free of any debris.**
-  **Do not allow any ferrous material to attach itself to the charging surfaces.**
-  **Keep the magnetic surfaces away from materials that may be adversely affected by their proximity to strong magnetic fields (e.g. watches, pacemakers, bank cards or magnetically stored media).**
-  **Prevent build up of debris on the charging surfaces as this may prevent charging or damage the charger. Disconnect the charger from the mains socket and remove any debris before reconnecting the charger to the device.**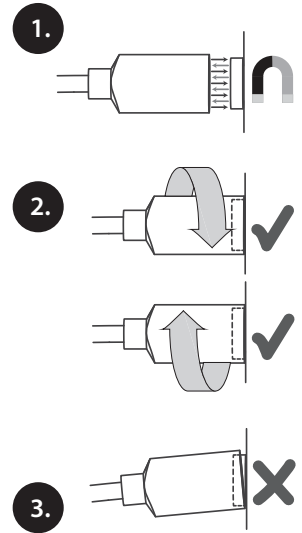

6.2 Battery Status

To check the battery condition press and **hold** the [-] button on the knee, one short beep will be heard and the LED will indicate the battery condition; see table below.

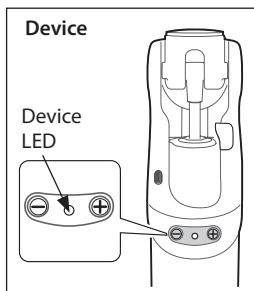Battery Charge Indicator			
	LED	Beeps	Battery Condition
Green	3 flashes	--	fully charged
Green	2 flashes	--	good
Green	1 flash	--	ok
Yellow	slow flash	8 x Short beeps	low charge
Red	slow flash	5 x Long beeps	no charge

When fully charged the device's battery is intended to provide up to 3 days of normal use (conditional on the type of use). We recommend that the knee be fully charged every night. Switching off the device when not in use will also prolong the period of use

Note... The device will take 30 seconds to reset after it is switched back ON [I]; refer to Section 6 Battery Charging (continued) for the reset sequence.

7 User Modes

The device has 2 user modes. Please note that these modes are only available if they have been enabled by your clinician.

Warning:

Care must be taken to only use those modes as instructed by your clinician.

1 Flexion Lock

Sets the limit of flexion to a predetermined angle.

The maximum lock angle is 45 degrees from fully extended.

2 Cycling (Free swing)

Releases stance resistance, the limb will then swing freely.

Warning:

In Cycling mode there is no stance control active, this mode must be de-activated before resuming walking to avoid falling and possible injury.

Flexion Lock mode

User activation of Flexion Lock mode with [-] button on the knee module

To activate Flexion Lock:

Fully extend the knee.

Press the [-] button 3 times, a short beep confirms each button press. **1.0B** 6 or **1.0N+** 3 medium beeps confirm that Flexion Lock is ON.

To de-activate Flexion Lock:

Press and hold [-] you will hear 2 long beeps.

Cycling mode

Warning:

In cycling (free swing) mode there is NO stance resistance.

User activation of Cycling mode with [-] button on the knee module

To activate Cycling mode:

Flex the knee to 90 degrees. Press the [-] button 4 times, a short beep confirms each button press. **1.0B** 5 or **1.0N+** 4 medium beeps confirm that Cycling mode is activated.

To de-activate Cycling mode:

Press and hold [-] you will hear 2 long beeps.

This device complies with part 15 of the FCC Rules.
Operation is subject to the following three conditions:-
1) This device may not cause harmful interference.
2) This device must accept any interference received, including interference that may cause undesired operation.
3) Contains pre-approved **Bluetooth®** module:
FCC ID: PVH0946 IC: 5325A-0946

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for any adverse outcome caused by any component combinations that were not authorized by them.

CE Conformity

This product meets the requirements of the European Regulation EU 2017/745 for medical devices. This product has been classified as a class I device according to the classification rules outlined in Annex VIII of the regulation. The EU declaration of conformity certificate is available at the following internet address: www.blatchford.co.uk

Medical Device

Single Patient – multiple use

Warranty

This device is warranted for 36 months, the foot shell for 12 months, and the glide sock for 3 months.

Extended warranties of an additional 12, 24, or 36 months are available.

A service between 30 and 36 months is required for all extended warranties.

For the maximum 36-month extended warranty, a second service must be carried out between 54 and 60 months.

We reserve the right to void extended warranties if the device is not serviced at the required intervals.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions.

The above may vary by market; consult your local representative for details.

See the Blatchford website for the current full warranty statement.

Reporting of Serious Incidents

In the unlikely event of a serious incident occurring in relation to this device it should be reported to the manufacturer and your national competent authority.

Environmental Aspects

This symbol indicates that the product contains electrical/electronic components and/or batteries that should not be disposed of in general waste or be incinerated at the end of the product's life.

At the end of the product's life, all electrical/electronic components and/or batteries should be recycled or disposed of in accordance with the current regulations for the handling of WEEE (Waste Electrical and Electronic Equipment), or equivalent local regulations. The remainder of the product should also be recycled where possible in accordance with local waste recycling regulations.

To help prevent potential harm to the environment or to human health from uncontrolled waste disposal, Blatchford offers a take-back service. Please contact Customer Services for details.

Trademark Acknowledgements

Linx and Blatchford are registered trademarks of Blatchford Products Limited.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG Inc., and any use of such marks by Blatchford is under License. Other trademarks and trade names are those of their respective owners.

Manufacturer's Registered Address

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© Blatchford Products Limited 2020. All rights reserved.
Endolite India Ltd.