Comfort Cushion Liner CTTCP22L–CTTCP40L
Comfort Locking Liner CTTLP22L–CTTLP40L

EN	User Guide	1
FR	Guide de l'Utilisateur	4
DE	Bedienungsanleitung für Anwender	7
IT	Guida per il Paziente	10
ES	Guia de Usuario	13
NO	Brukerhåndbok	16

Endolite Liner

endolite

Comfort Cushion Liner CTTCP22L - CTTCP40L Comfort Locking Liner CTTLP22L - CTTLP40L

Please keep these instructions.

The Endolite Comfort silicone liner provides a cushioned socket interface and is manufactured from biocompatible materials.

 **Wearers with sensitive skin, diabetics and vascular cases should be extra vigilant and may need to apply lubricant to sensitive areas. We recommend a routine visual check and if required consult your medical practitioner.**

 Take care when handling the Liner to avoid possible contamination with materials such as fibreglass which will stick to the Liner and cause skin irritation.


Donning and doffing

 Care must be taken during donning/doffing not to damage the liner with fingernails or sharp jewelry.

 Do **NOT** pull or stretch the liner.

1. **Roll** the liner inside out.
2. Make sure the liner is positioned correctly to suit the top profile.
3. **Roll** the liner on taking care to expel any trapped air.

 Warning: when donning a sock, clothing and the prosthetic limb, be aware that the liner can build up a static charge.

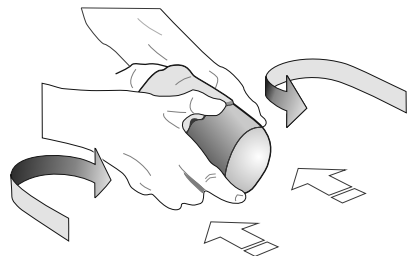Cleaning and care

Cleaning and care

Wash the inside of the liner daily to avoid any build up of bacteria. Use a simple, unperfumed soap (ideally pH balanced). Hand wash or Cool machine wash.

Rinse well in clean water to remove all residues and dry thoroughly before use.

Take care when drying and handling the liner. Pat dry with a lint-free cloth or leave to air dry. Do **NOT** leave the liner inside out to dry as this can stretch the liner and permanently distort the shape.

Take care handling the liner when it is inside out to avoid picking up dust, grit and other contamination which could cause irritation to the skin.

Skin care

It is recommended that when using the liner the residuum should be washed daily with a simple, unperfumed soap (ideally pH balanced). Rinse thoroughly to remove any soapy residue before drying thoroughly. Use a lotion for dry skin. Make sure that any damaged skin is properly and suitably dressed.

DO **NOT** tumble dry.

Do **NOT** use alcohol sprays, household cleaners or abrasives which could damage the liner and cause skin irritation.

Precautions

Warning: To avoid danger of suffocation, keep the Liner away from babies and children

Keep away from direct heat sources which could damage the Liner.

Stop using the liner if it is torn or damaged.

For other medical conditions please check with your physician or medical practitioner for following this recommendation for skin care.

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

CE Conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty:

The liner is warranted for 6 months from original date of purchase unless otherwise stated. See Endolite catalogue for details.

Environmental Aspects

This product is fabricated from silicone rubber and fabric that cannot be easily recycled: please dispose of it responsibly as general waste, as per local waste handling regulations.

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Endolite India Ltd
Endolite Deutschland GmbH,