Silcare Breathe Cushion Liner

Instructions for Use

SBTTCP22L - SBTTCP40L

EN	Instructions for Use	2
FR	Instructions D'Utilisation	6
DE	Gebrauchsanweisung	10
IT	Istruzioni per L'Uso	15
ES	Instrucciones de Uso	20
NO	Bruksanvisning	24

Blatchford:

1 Description and purpose

EN

Application:

These instructions are for use by the practitioner.

The Silcare Breathe perforated silicone liner provides a perforated cushioned socket interface which allows moisture to pass through the perforations keeping the skin dry. It is manufactured from biocompatible materials.

The liner is suitable for users of all activities, although is best suited to users who have the potential to achieve activity levels 3 and 4

Please read these instructions carefully before fitting the liner.

To optimise comfort ensure that the user is instructed in the correct way to handle and don/doff the liner. Also ensure that they are made aware of how to maintain the liner and to keep it clean for maintaining hygiene as set out in these instructions.

The user should be advised to contact their practitioner if their condition changes.


2 Product selection and fitting

To determine the correct size

It is important that the correct size is specified to ensure proper fit and comfort in use.

Measure the circumference 4cm up from the distal end. If an exact size is measured choose the next size down.

Sizing example:

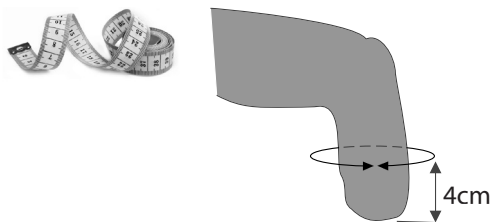After donning the liner allow the patient to wear the liner for approximately 10 minutes. If any numbness or tingling is felt remove the liner and when normal sensation returns try the liner again if numbness occurs again do not use the liner.

If the measured size is an exact match for a liner size choose the next smaller size to ensure a snug fit for the liner.

Wearers with sensitive skin, diabetics and vascular cases should be extra vigilant and may need to apply lubricant to sensitive areas. We recommend a routine visual check and if required the user should consult their medical practitioner.

Size (cm)									
22	23.5	25	26.5	28	30	32	34	36	40
✓	✓	✓	✓	✓	✓	✓	✓	✓	✓

Order Example:

To specify liner model:

e.g. *SBTTCP26L*

Silcare Breathe [SB]

Trans tibial [TT]

Cushion [C]

Parallel [P]

Size 22-40

Light Tone [L]

Silcare
Breathe

Trans
Tibial

Cushion

Parallel

Size
26.5

Light
Tone

Trimming

The liner may be trimmed to suit as required.

We recommend the use of a rotating disc type cutter to achieve a 'rolled' or bevelled edge.

Trim the liner to allow a suitable overlap beyond the hard edge of the socket. The liner may be trimmed in a 'waveform' profile to minimise the risk of edge effect.

Where possible try to avoid abrupt changes of contour and sharp edges which could cause tears in the silicone and fabric.

Take care when handling the liner to avoid possible contamination with materials such as fibreglass which will stick to the liner and cause skin irritation.

Donning and doffing

Care must be taken during donning/doffing not to damage the liner with fingernails or sharp jewelry.

Do **NOT** pull or stretch the liner.


1. **Roll** the liner inside out.
2. Make sure the liner is positioned correctly to suit the top profile.
3. **Roll** the liner on taking care to expel any trapped air.

Warning: when donning a sock, clothing and the prosthetic limb, be aware that the liner can build up a static charge.

The fabric cover may ladder slightly around the perforations. This will not affect the function or effectiveness of the liner.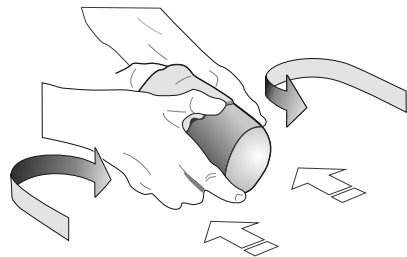

3 Cleaning and care

Wash the inside of the liner daily to avoid any build up of bacteria. Use a simple, unperfumed soap (ideally pH balanced).

Hand wash or cool machine wash.

Perforated liners should be filled with warm water, the proximal (open end) held closed and the liner gently squeezed to clear /flush the perforations.

Rinse well in clean water to remove all residues and dry thoroughly before use.

Take care when drying and handling the liner. Pat dry with a lint-free cloth or leave to air dry. Do **NOT** leave the liner inside out to dry as this can stretch the liner and permanently distort the shape.

Take care handling the liner when it is inside out to avoid picking up dust, grit and other contamination which could cause irritation to the skin.

Skin care

It is recommended that when using the liner the residuum should be washed daily with a simple, unperfumed soap (ideally pH balanced). Rinse thoroughly to remove any soapy residue before drying thoroughly. Use a lotion for dry skin. Make sure that any damaged skin is properly and suitably dressed.

Do **NOT** tumble dry.

Do **NOT** use alcohol sprays, household cleaners or abrasives which could damage the liner and cause skin irritation.

Note:

Perspiration may discolour some socket materials.

Precautions

Warning: To avoid danger of suffocation, keep the liner away from babies and children.

Keep away from direct heat sources which could damage the liner.

Stop using the liner if it is torn or damaged.

Stop using the liner if perforations become enlarged as this can cause skin to become trapped and blistered.

For other medical conditions please check with your physician or medical practitioner for following this recommendation for skin care.

Any deterioration in residuum condition should be reported to your practitioner.

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

CE Conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty:

The liner is warranted for 6 months.

See Blatchford website for the current full warranty statement

Manufacturer's Registered Address

Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH, UK.

Silcare Breathe and Blatchford are registered trademarks of Blatchford Products Limited.

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Blatchford SAS
Endolite India Ltd